

AD2 Magnetic Mounting System Installation


Thank you for your purchase of the AD2 Magnetic Mounting System. The instructions below may be used for AD2 articulators and MCDs as well as all Panadent articulators. While the pictures shown are on the top frame of the instrument, the same procedure is used to mount on the bottom frame as well.




1) Remove the mounting knob by pulling up and rotating counterclockwise until the threaded screw clears the frame.



2) Hold mounting knob so that the original nylon washer can be removed.

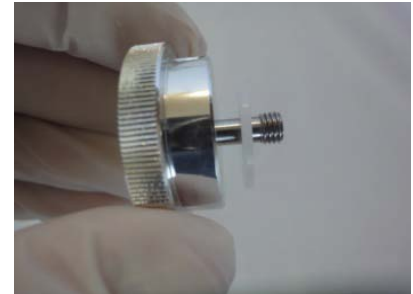

3) Install the AD2 EZ Cushion nylon washer supplied with your magnetic mounting system.

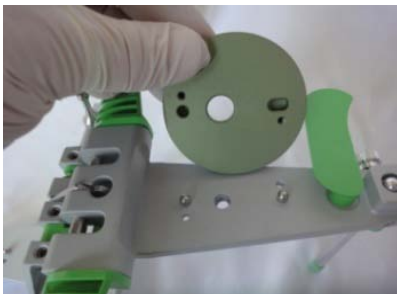

4) Position the magnetic base over the frame, noting the position of the index holes.



5) Place the magnetic base on the frame.



6) Position the allen head screws (supplied) on the frame so they can be tightened.



7) Tighten the allen screws into the magnetic base until snug.



8) Replace the mounting knob (with the nylon EZ Cushion washer) by threading into the frame and rotating clockwise.



9) Position the magnet and retainer cup (supplied) over the magnetic base.



10) Tighten the mounting knob into the magnet retainer cup until snug, ensuring that the surface of the magnet sits flush with the magnetic base.



11) At this point, your AD2 magnetic mounting system should look like this. Repeat these steps to install the remaining magnetic base.



12) Once the magnetic mounting system has been installed completely as shown, your AD2 instrument is ready for use.