

Advanced Dental Designs, Inc.

9880 Indiana Avenue, #4 • Riverside, CA 92503 • USA

USA & Canada: 800.232.2849 International: +001 951.579.4535

www.ad2usa.com

 ϵ

Threaded Insert Reset Instructions

Item #AR130103

1



Remove mounting knob from articulator frame by unthreading it (counterclockwise). Note the small allen screw in the knob.

2



Using the 2.5mm hex screwdriver from your articulator, loosen the small allen screw in the mounting knob and remove the threaded insert. Note the notched section on the threaded insert

3



Locate the threaded insert back into the mounting knob, ensuring that the notched section is aligned with the allen screw. When ready, push insert all the way into the mounting knob and tighten allen screw to firmly hold threaded insert.