Advanced Dental Designs, Inc.
 22640 Goldencrest Drive, #106
 Moreno Valley, CA 92553 • USA
 USA & Canada: 800.232.2849
 International: +001 951.579.4535
 www.ad2usa.com

MedEnvoy Global B.V.
 Prinses Margrietplantsoen 33 - #123
 2595 AM The Hague The Netherlands

Delar Wax Instructions

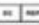
Delar Registration Wax (REF: DL600008, DL600013, DL600015, DL600016, DL600021, DL600023, DL600024, DL600025, DL600026, DL600028, DL600030)

Delar Bite Registration Wax was designed specifically for obtaining jaw relation records. Heat in thermostatically controlled water bath at 58° C (135° F) for 1 minute. After obtaining jaw relation records, place in cold water to regain maximum hardness quickly and to facilitate trimming off excess wax. For Single Use Only. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

 Advanced Dental Designs
 22640 Goldencrest Dr, #106
 Moreno Valley, CA 92553
 (800) 232-2849

MedEnvoy Gloval B.V.
 Prinses Margrietplantsoen 33, Suite 123
 2595 AM The Hague
 The Netherlands

 LOT	Batch code	 Manufacturer
 REF	Catalog #	 EU authorized rep

Item #PK970022

Doc Revision#: L-DELAR REV B 02/15/24

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Advanced Dental Designs
 22640 Goldencrest Dr, #106
 Moreno Valley, CA 92553
 (800) 232-2849

MedEnvoy Gloval B.V.
 Prinses Margrietplantsoen 33, Suite 123
 2595 AM La Haya
 Países Bajos

LOT	Batch code		Manufacturer
REF	Catalog #		EU authorized rep

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 Advanced Dental Designs
 22640 Goldencrest Dr, # 106
 Moreno Valley, CA 92553
 (800) 232-2849

MedEnvoy Gloval B.V.
 Prinses Margrietplantsoen 33, Suite 123
 2595 AM La Haye
 Pays-Bas

LOT	Batch code		Manufacturer
REF	Catalog #		EU authorized rep

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Advanced Dental Designs
22640 Goldencrest Dr, #106
Moreno Valley, CA 92553
(800) 232-2849

MedEnvoy Gloval B.V.
Prinses Margrietplantsoen 33, Suite 123
2595 AM L'Aia
Paesi Bassi

LOT	Batch code		Manufacturer
REF	Catalog #		EU authorized rep

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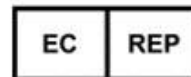
Advanced Dental Designs
22640 Goldencrest Dr, 106
Moreno Valley, CA 92553
(800) 232-2849

MedEnvoy Gloval B.V.
Prinses Margrietplantsoen 33, Suite 123
2595 AM The Hague
Países Baixos

LOT	Batch code		Manufacturer
REF	Catalog #		EU authorized rep

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
Advanced Dental Designs
22640 Goldencrest Dr, #106
Moreno Valley, CA 92553
(800) 232-2849

MedEnvoy Gloval B.V.
Prinses Margrietplantsoen 33, Suite 123
2595 AM Haga
Olanda

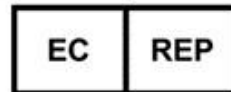LOT	Batch code	Manufacturer
REF	Catalog #	EU authorized rep

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