## EZ Bow Instructions

### Items #AR540250, AR560150 – AR560950

1 Install laser, bubble cube and index tray on the EZ Bow, making sure that the index tray is secure and flat.

2 Evaluate patient for canted smile or steep plane of occlusion. Apply more bite reg material to the high areas to capture bite registration. Do not rotate the EZ bow! Keep it flat and aligned on the facial midline


3 Apply wax tabs or bite registration compound to the index tray in molar and bicuspid areas (9, 11, 1, and 3 o'clock positions).


4 Patient should be sitting or standing upright, with head level in preparation for the EZ Bow. Please note that The laser diode alignment accessory is low power with an output power of 1MW and wavelength of 658nm.

5 Position the bow in front of the face to help identify sagittal and anterior-posterior level. Have patient close eyes, turn laser on and position on facial midline. Patient can now open eyes. Caution: Do not stare in laser beam. Avoid direct eye exposure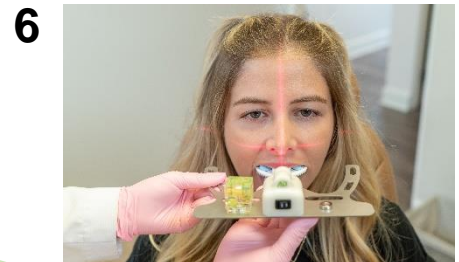

6 Insert EZ Bow into mouth, raise upward into maxilla, with central incisors pressed against front vertical tab. EZ Bow should be supported by doctor's hands. Do not have patient bite onto bow.

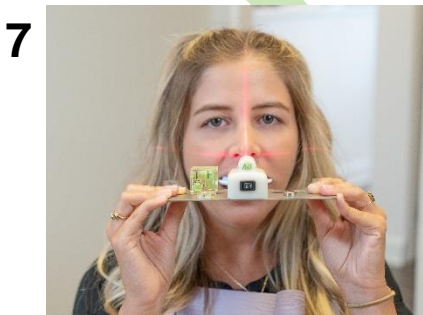7 Once complete, have the patient support the bow with fingers. Step back and take photographs as needed.

8 Remove the bow from the patient's mouth and remove the index tray from the bow. Write patient's name on the underside of the index tray. The index tray can be removed by gripping the sides of the tray and pulling.

9 Alternatively, the index tray can be removed by placing on flat surface and applying light downward pressure on the EZ Bow.

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