

EZ Bow Instructions

Items #AR540250, AR560150 – AR560950



1 Install laser, bubble cube and index tray on EZ Bow, making sure that index tray is secure and flat.



2 Evaluate patient for canted smile or steep plane of occlusion. Apply more bite reg material to the high areas to capture bite registration. Do not rotate the EZ bow! Keep it flat and aligned on the facial midline




3 Apply wax tabs or bite registration compound to index tray in molar and bicuspid areas (9, 11, 1, and 3 o'clock positions).




4 Patient should be sitting or standing upright, with head level in preparation for the EZ Bow.



5 Position the bow in front of the face to help identify sagittal and anterior-posterior level. Have patient close eyes, turn laser on and position on facial midline. Patient can now open eyes.

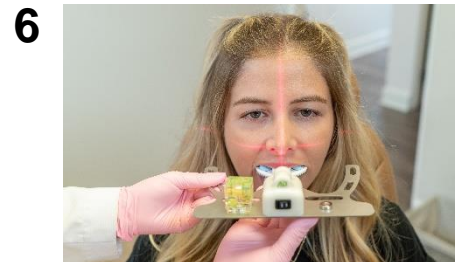

6 Insert EZ Bow into mouth, raise upward into maxilla, with central incisors pressed against front vertical tab. EZ Bow should be supported by doctor's hands. Do not have patient bite onto bow.

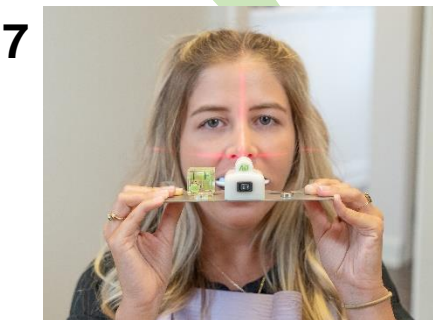

7 Once complete, have patient support bow with fingers. Step back and take photographs as needed.



8 Remove bow from patient's mouth and remove index tray from bow. Write patient's name on underside of index tray. Index tray can be removed by gripping sides of tray and pulling.



9 Alternatively, index tray can be removed by placing on flat surface and applying light downward pressure on EZ Bow.